

WOUND PREPARATION

Wound Bed Preparation Begins with Effective Cleansing...

NovaGran™ Antimicrobial Wound Solution

- 510(k) cleared for OTC
- 0.018% HOCl in saline
- Non-irritating
- Zero toxicity
- Antimicrobial

NovaGran Antimicrobial Wound Solution with hypochlorous acid (HOCl) is for the management of minor skin abrasions, minor lacerations, minor irritations, minor cuts, minor burns, and intact skin.

Item #	Product Size	HCPC	Packaging
1420	2 oz.	A6260	9-pack
1422	8 oz.	A6260	6-pack
1423	16 oz.	A6260	6-pack



Complete wound care From Cleansing to Closure™

Complimentary samples are available for medical professionals to evaluate the NovaGran System of Wound Care Products, and **experience the SEA Change**. During the evaluation, we will partner with you by providing samples and support materials for one or more patients until the conclusion of the evaluation (trial). ***Simply call or visit our website today!***

NovaGran™
Wound Care
Products

1-888-980-9702 www.NovaGranProducts.com

Indication:

For management of minor skin abrasions, minor lacerations, minor irritations, minor cuts, minor burns, and intact skin.

Directions for Use:

Apply NovaGran™ Antimicrobial Wound Solution directly on and around the wound to flush away dirt and debris. Thoroughly clean the wound. If dressing is used, saturate it with NovaGran Antimicrobial Wound Solution and apply directly over the wound. Dressing should extend beyond the wound margins. Flush the wound with NovaGran Antimicrobial Wound Solution and apply a fresh moistened dressing one or more times daily as needed. Soak adherent dressings with NovaGran Antimicrobial Wound Solution to ease removal.

Warning:

Not for Injection. In the case of deep or puncture wounds, animal bites, or serious burn, consult a healthcare professional. If conditions persist or worsen for longer than seven days, consult a healthcare professional. For external use only, keep out of the reach of children.

Not for Infusion**Handling:**

No special precautions required.

Storage:

Store at room temperature in its original sealed container away from direct sunlight and heat.

Do not freeze.

Do not use after expiration date.

The logo for NovaGran Wound Care Products features the brand name "NovaGran" in a large, bold, teal font. The letter "o" in "Nova" is stylized with three curved lines underneath it. To the right of "NovaGran" is a small "TM" trademark symbol. Below the brand name, the words "Wound Care" and "Products" are stacked in a smaller, black, sans-serif font.

NovaGranTM
Wound Care
Products

1-888-980-9702 **www.NovaGranProducts.com**